

SOLUSCOPE CLN



PH NEUTRAL DETERGENT

PROVEN EFFICACY
AGAINST BIOFILM
AND PROTEINS

COMPLIANT WITH
EN ISO 15883-4

COMPATIBLE WITH
A LARGE RANGE
OF FLEXIBLE
ENDOSCOPES

LOCK KEY SYSTEM
AND CANNULA
INTEGRATED TO THE
CONTAINER

INDICATIONS

- Detergent for flexible endoscopes.
- To be used exclusively with *Soluscope Serie 4 PA*.

COMPOSITION

- Association of surfactants with controlled foam
- Sequestering agent
- Excipients

CHARACTERISTICS

- Pale yellow clear liquid
- Density at 20°C: 1.02mg/ml
- pH pure product at 20°C: about 6.9*
- pH diluted at 1%: about 7.8*
- Conductivity at 0.5%: 315µS/cm

* pH value measurements under operating conditions always by means of a pH meter with temperature compensation

USER MANUAL


- Put the container in place.
- Screw the white cap connected to your *Soluscope Serie 4 PA* onto the container (refer to the Washer-Disinfector user manual).
- Make sure that the vent hole is not blocked.
- Automated dilution: 1% (10mL/L), 35°C, 90 sec.
- Shelf life after opening: 2 months while connected to the Washer-Disinfector (write the opening date on the label in the location provided for this purpose).

MEDICAL DEVICE USE SAFELY

- Please read label and product information.
- Medical Device.

STORAGE

Dangerous – respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products).

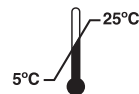

PACKAGING

- 2 containers of 5L



D0737E-EN



SOLUSCOPE S.A.S
(An Ecolab company)
100, rue du Fauge
Z.I. Les Paluds - 13400 Aubagne
France



www.ecolab.com



www.soluscope.com