


# SOLUSCOPE C+

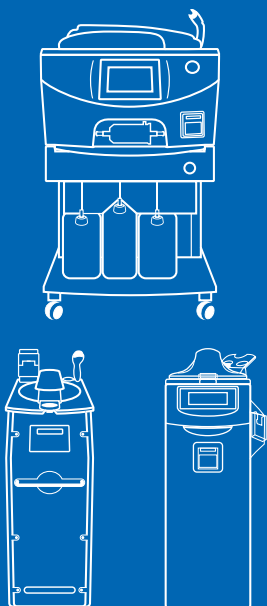

OPTIMAL EFFICACY ON  
BACTERIAL AND YEAST  
GERMS AND BIOFILMS

COMPLIANT WITH  
EN ISO 15883-4

COMPATIBLE WITH  
A LARGE RANGE OF  
FLEXIBLE ENDOSCOPES

SPECIFIC CONNECTION  
AVOIDING CONFUSION  
WITH THE ADDITIVE  
AND THE DISINFECTANT  
OF THE WASHER-  
DISINFECTOR

## INDICATIONS

- Pre-disinfectant detergent for flexible endoscopes.
- To be used exclusively with *Soluscope Serie 3, Sprint, Sprint PT, Serie ENT and Serie TEE*.

## COMPOSITION

- Quaternary ammonium
- Glycolic acid
- Tensioactives

## CHARACTERISTICS

- Clear, blue solution
- Density: between 1.00 and 1.03
- pH (pure product): between 1.50 and 3.00
- Stability: 36 months (closed), 2 months (opened)

## USER MANUAL


- Screw the plunger, equipped with the blue cap, onto the bottle.
- Make sure that the vent hole is not blocked.
- Put the bottle in place.
- Concentrated solution for professional use.

## MEDICAL DEVICE USE SAFELY

- Please read label and product information.
- Medical Device.

## STORAGE

Dangerous – respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products).

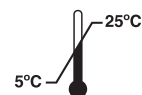

## PACKAGING

- 2 containers of 5L



SERIE 3 PA version



SERIE ENT / SERIE TEE

D0775A-EN



**SOLUSCOPE S.A.S**  
(An Ecolab company)  
100, rue du Fauge  
Z.I. Les Paluds - 13400 Aubagne  
France



[www.ecolab.com](http://www.ecolab.com)



[www.soluscope.com](http://www.soluscope.com)